

FABRICATION OF A NEW PROSTHESIS WITH PRECI-HORIX

1. Take a full arch impression. Measure the length of the pattern in the patient's mouth.
2. Reduce the **1707** pattern analogue to this length and provide it with retention for the plaster model.
3. Seat the analogue in position in the impression and pour a model in stone or epoxy material.
4. Place the **1703** space maintainers on the pattern analogue and block out the rest of the bar with a solid material (plaster, silicone, or cement).
5. Provide the space maintainers with the **1706** housing (recommended) and finish the prosthesis in the normal manner.
6. Remove the space maintainers with a sharp instrument.
7. Seat the desired **1802/1805/1806** riders with the supplied **1705** insertion tool in position (see INFO 016).