

REBASING/RELINING A PROSTHESIS WITH PRECI-BAR

Article numbers:	1100/H/MR/OR	1100/H/MR/PA	1100/H/MR/IN
	1102/H/MR/OR	1102/H/MR/PA	1102/H/MR/IN
	1103/B/OR	1103/B/PA	1012/B
	1103/BL/OR	1103/BL/PA	
	1101/B/OR	1101/B/PA	1013/B
	1101/BL/OR	1101/BL/PA	
	1100/B/OR	1100/B/PA	1011/B
	1100/BL/OR	1100/BL/PA	

1. Take a reline impression with the rider in the prosthesis.
2. Fill up the rider and the bar area with silicone.
3. Pour a stone model. Always use a reline flask or jig.
4. Reline as usual.
5. Finish and polish.
6. Activate the rider if necessary (see INFO 108).