

FABRICATION OF A NEW PROSTHESIS WITH PRECI-CLIP

1. Block out the undercuts of the bar and take a full arch impression.
2. Pour a stone model. The entire bar area must be made of epoxy material or self-curing acrylic resin.
3. Place the **1105/1106/1125/1126** rider with the space maintainer on the model.
4. Block out the entire bar area, leaving the riders for 2/3 free. Also cover the top of the bar with wax to create resilience.
5. Also block out the lower part of the riders to allow for future activation.
6. Finish the prosthesis as usual.
7. After polymerization, remove the space maintainer and activate the riders if necessary (see INFO 107).