

## REPLACING A RIDER IN A PROSTHESIS WITH PRECI-CLIP

1. Cut a length from the **1707** pattern analogue to the length of a rider. Provide the underside of the analogue with retention for fixation in the stone model.
2. Seat the analogue with the space maintainer in the rider in the prosthesis and pour a stone model.
3. Cut the old rider in the prosthesis free and remove it.
4. Place the new **1105/1106/1125/1126** rider with the space maintainer on the model and polymerize it with self-curing acrylic resin.
5. Activate the rider after finishing if necessary (see INFO 107).