

## FABRICATION OF A PROSTHESIS WITH PRECI-BALL

1. Take a full arch impression.
2. Place the **1201 D** model analogue in the impression and pour a stone model.
3. Place the **RA 0055** large space maintainer over the male on the post-coping. Fill up the area between papilla and post-coping with plaster. The space maintainer must cover the edge of the papilla by 1 to 1.5 mm.
4. Fill the female half-way with silicone and seat it over the male. The pvc ring must surround the female segments. Leave the retention zone for acrylic resin on the female free.
5. Place the wax setup on the model. Prepare for investing.
6. Polymerize and finish.
7. Remove the space maintainer after polymerization. Do not remove the pvc ring.
8. Check the retention and activate the female if necessary (see INFO 053).