

## REPLACING A FEMALE IN A PROSTHESIS WITH PRECI-BALL

1. Press the **1201 D** model analogue into the female to be replaced in the prosthesis.
2. Pour a stone model.
3. Cut the old female out of the prosthesis. Make a lingual opening in the prosthesis.
4. Fill the new **1201 A**, **1205 A** or **1207 A + 694 AKS2** female half-way with silicone and seat it over the model analogue. Use the space maintainer. The pvc ring must surround the female segments. Leave the retention zone for acrylic resin on the female free.
5. Seat the prosthesis on the model and let self-curing acrylic resin flow in.
6. Polymerize and finish.
7. Do not remove the pvc ring.
8. Check retention and activate the female if necessary (see INFO 053).