

NEW PROSTHESIS WITH BONDED FEMALE

If the abutment retainers have been replaced or if there are problems with the prosthesis, the fabrication of a new prosthesis may be necessary.

CHAIRSIDE PROCEDURE

1. Assemble the **RE H 14 (M2)** or **H 14 (M3)** impression tool with the **RE 0031 (M2)** or **694 C (M3)** spring pin.
2. Place this combination in the female in the mouth.
3. The combination must be absolutely stable and secure in the female. If not, activate the spring pin (see INFO 059).
4. Block out the undercuts of the female with soft wax.
5. Take impression with a custom-made impression tray.

LABORATORY PROCEDURE

1. Assemble the above-mentioned impression tool with the spring pin, the space maintainer and the **RE H 13 (M2)** or **H 13 (M3)** model accessory.
2. Place this combination in the impression.
3. Pour a model in stone which incorporates the model analogues.
4. Make a new secondary construction.