

REPLACING A PRECI-CLIX FEMALE

PRECI-CLIX attachments are supplied with the **1231** yellow standard females.

When the retention of the female no longer meets the requirements, it can be replaced with the **1232** white female with reduced retention or the **1233** red female with increased retention.

1. Bend the lips of the female with a sharp instrument towards the inside.
2. Carefully drill with the **IMP-XH-135** head-cutting burr a hole in the centre of the female (up to the housing).
3. Remove the female with a sharp instrument.
4. Position the desired female on the **1222** insertion tool and press this assembly into the housing.
5. Check if the required retention is attained.