

## REBASING/RELINING A PROSTHESIS WITH PRECI-CLIX

### CHAIRSIDE PROCEDURE

1. Block out the attachments with soft wax.
2. Press the **1251 B** space maintainer over the male in the mouth.
3. Apply self-curing acrylic stops to the tissue side of the prosthesis in the area of the retromolar pad or the tuber maxillae.
4. Bring the patient's jaws manually in centric relation and hold this position while the self-curing acrylic resin is hardening.
5. Apply impression material to the prosthesis and take impression. The spring pin must snap in correctly.

### LABORATORY PROCEDURE

1. Remove any impression material from the female.
2. Press the **1201 D** model analogue or the **RE H 14 + 1206 C** assembly into the females in the prosthesis.
3. When using the **RE H 14 + 1206 C** assembly, fill up the space between the **RE H 14** accessory and the metal sleeve with wax.
4. Pour a stone model and prepare it for relining.
5. Always use the space maintainer during processing.
6. Reline as usual.
7. Replace the female after finishing and polishing if necessary (see INFO 122).