

REPLACING A PRECI-SAGIX FEMALE

All PRECI-SAGIX introduction kits are supplied with females with 3 different levels of retention.

When the retention of the female does not meet the requirements, it can be replaced with a **1312** (Mini \varnothing 1.7 mm) or **1362** (Standard \varnothing 2.2 mm) white female with reduced retention, a **1311** (Mini \varnothing 1.7 mm) or **1361** (Standard \varnothing 2.2 mm) yellow female with normal retention, or a **1313** (Mini \varnothing 1.7 mm) or **1363** (Standard \varnothing 2.2 mm) red female with increased retention.

1. Use a pointed instrument to remove the female.
2. Steam clean the cavity in the frame.
3. Insert the female with the desired level of retention using the **1323** universal insertion tool (for \varnothing 1.7 mm and 2.2 mm).
4. Check if the female is properly seated and if the desired retention is obtained.