

VITA PROSTHETIC SOLUTIONS

Mould Chart



For the ideal dental prosthetics – natural, reliable, rich in variation.

It's so simple – three steps to the correct tooth mould!

- 1. Select the category/
tooth line**
- 2. Select the mould
group**
- 3. Determine the
appropriate tooth mould**

Decide on a category/tooth line based on the requirements of the case and indication.

Select the appropriate mould group for the patient's face and jaw mould.

Determine the appropriate tooth mould based on the cuspid and laugh lines.

1. Select the appropriate category/tooth line

Vivid play of shape and shade for natural results.



Ideal variety of shade and moulds for all cases.





Solid, standard dentures with excellent cost efficiency.



Efficient CAD/CAM dentures for greater productivity.



> Recommended anterior / posterior combinations

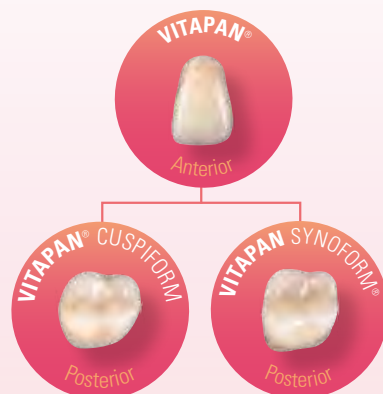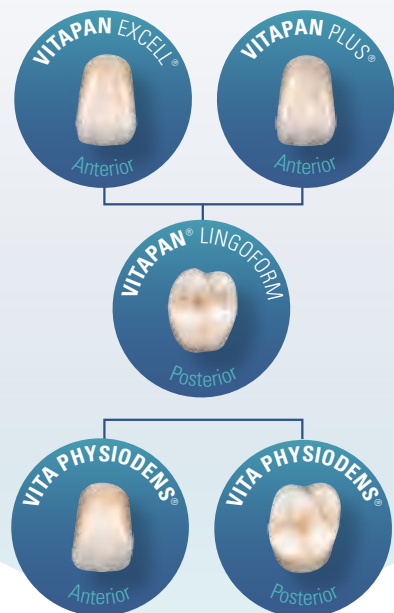


> Recommended indication(s)*

	★			⊕		☑		😊	
Anterior	VITAPAN EXCELL	VITAPAN PLUS	VITA PHYSIODENS	VITAPAN		VITA MFT	VITA VIONIC VIGO		VITA VIONIC FRAME
Posterior	VITAPAN LINGOFORM		VITA PHYSIODENS	VITAPAN CUSPIFORM	VITAPAN SYNOFORM	VITA MFT	VITA VIONIC VIGO		VITA VIONIC FRAME
Full dentures	●	●		●		●	●		●
Partial dentures	●	●	●	●		●			●
Hybrid dentures			●						
Implant-retained dentures	●	●							
Gerontopros-thetics					●				

* These are recommended indications for the respective prosthetic solutions/prosthetic teeth, but the entire spectrum of full/partial/composition and implant prostheses are typically possible!


VITAPAN EXCELL	VITAPAN PLUS	VITAPAN LINGOFORM	VITA PHYSIODENS	VITAPAN	VITAPAN CUSPIFORM	VITAPAN SYNOFORM	VITA MFT	VITA VIONIC VIGO	VITA VIONIC FRAME
----------------	--------------	-------------------	-----------------	---------	-------------------	------------------	----------	------------------	-------------------

2. Select the appropriate mould group

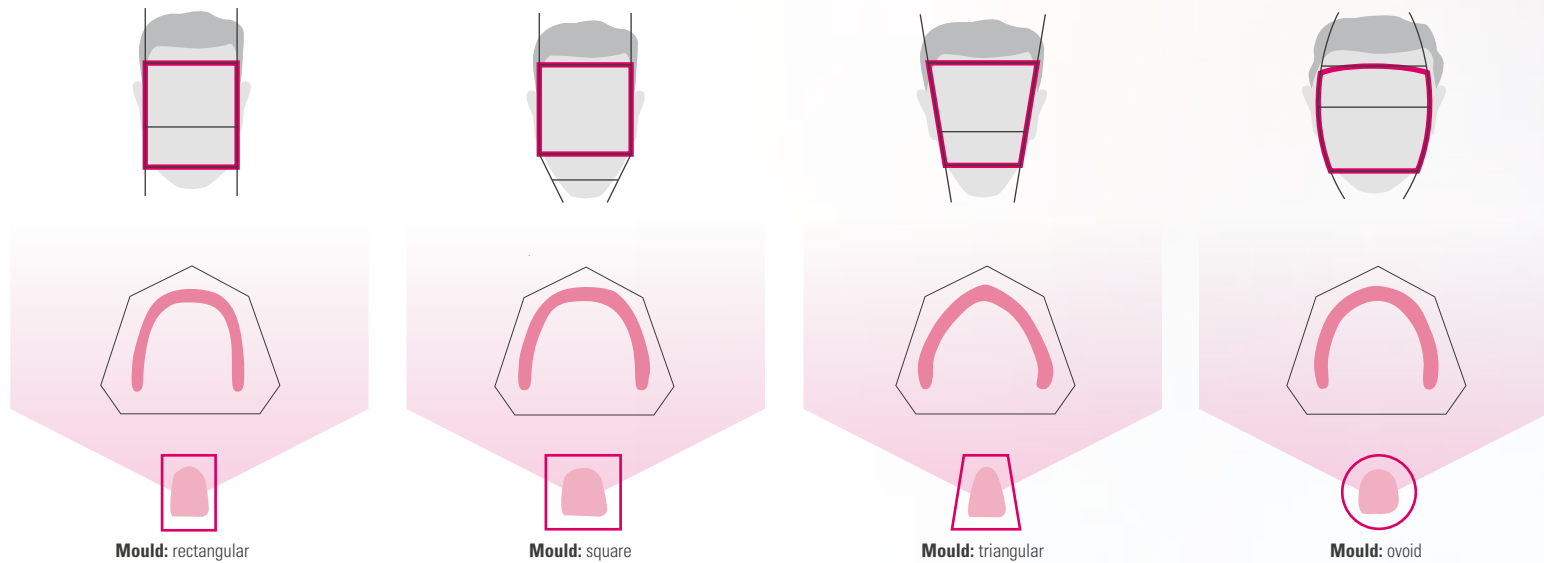


3. Determine the appropriate tooth mould

The required width and length of the anterior teeth are determined using the cuspid and the laugh line.

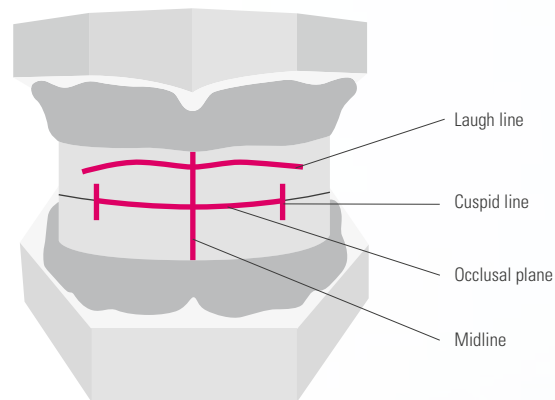

Laugh line

This line is the determining factor for the length of the upper anteriors. The tooth necks should be situated above this line.

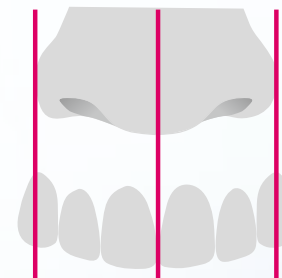

Cuspid line

This line is critical for the width of the upper anteriors. The tips of the upper cuspids should be aligned with this line.



VITAPAN
EXCELL

VITAPAN
PLUS

VITAPAN
LINGOFORM

VITA
PHYSIODENS

VITAPAN

VITAPAN
CUSPIFORM

VITAPAN
SYNOFORM

VITA
MFT

VITA VIONIC
VIGO

VITA VIONIC
FRAME

Upper Anteriors



ovoid

041 41.3 7.4 9.1	043 43.1 7.9 9.1	045 45.1 8.4 9.7	047 47.3 8.6 10.3
049 49.1 9.0 10.9	051 51.4 9.3 12.3		
T42 42.0 7.7 9.5	T44 44.1 8.1 9.9	T46 46.2 8.2 10.4	T47 46.9 8.5 10.9
T48 48.1 8.8 11.3	T50 50.0 9.2 11.6	T54 54.4 9.8 12.7	
R45 45.4 8.2 9.9	R47 47.0 8.6 11.2	R49 49.1 8.9 11.7	




triangular



rectangular

**S46**


46.3

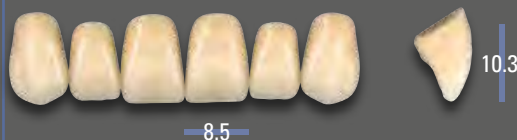

Lower Anteriors

L33

33.2

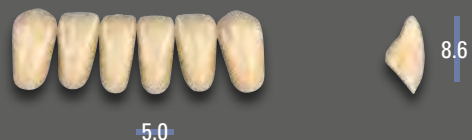**L35**

35.3

**L37**

37.3

**L39**

39.0

**L41**

41.2



► Note: For VITAPAN LINGOFORM moulds, please select the appropriate register

VITAPAN EXCELL Anterior	UJ / No.  mm	041 41.3	043 43.1	045 45.1	047 47.3	049 49.1	051 51.4	T42 42.0	T44 44.1	T46 46.2	T47 46.9	T48 48.1	T50 50.0	T54 54.4	R45 45.4	R47 47.0	R49 49.1	S46 46.3
	LJ / No.  mm	L33 33.2	L33 33.2	L35 35.3	L37 37.3	L39 39.0	L41 41.2	L33 33.2	L33 33.2	L35 35.3	L37 37.3	L37 37.3	L39 39.0	L41 41.2	L35 35.3	L37 37.3	L39 39.0	L35 35.3
VITAPAN LINGOFORM Posterior	UJ / No.  mm	21L 30.1	21L 30.1	22L 31.6	22L 31.6	23L 34.2	24L 35.8	21L 30.1	21L 30.1	22L 31.6	22L 31.6	22L 31.6	24L 35.8	24L 35.8	22L 31.6	22L 31.6	23L 34.2	22L 31.6
	LJ / No.  mm	21L 30.7	21L 30.7	22L 32.1	22L 32.1	23L 34.8	24L 36.4	21L 30.7	21L 30.7	22L 32.1	22L 32.1	22L 32.1	24L 36.4	24L 36.4	22L 32.1	22L 32.1	23L 34.8	22L 32.1


The information contained in the combination table is only intended as a guide, and can vary depending on the patient's individual situation.

VITAPAN EXCELL	VITAPAN PLUS	VITAPAN LINGOFORM	VITA PHYSIODENS	VITAPAN	VITAPAN CUSPIFORM	VITAPAN SYNOFORM	VITA MFT	VITA VIONIC VIGO	VITA VIONIC FRAME
-------------------	-----------------	----------------------	--------------------	---------	----------------------	---------------------	-------------	---------------------	----------------------



Upper Anteriors




ovoid

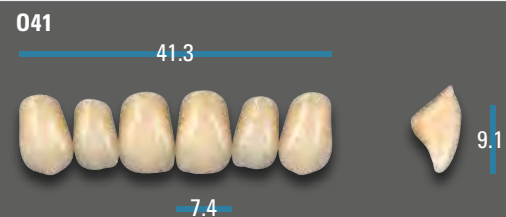

triangular

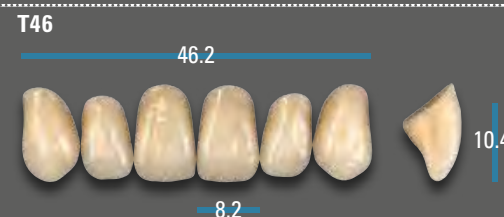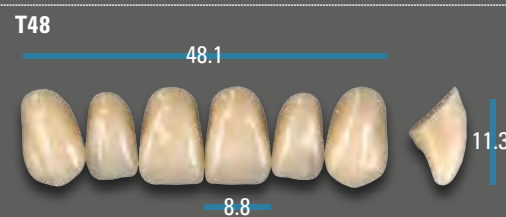

rectangular

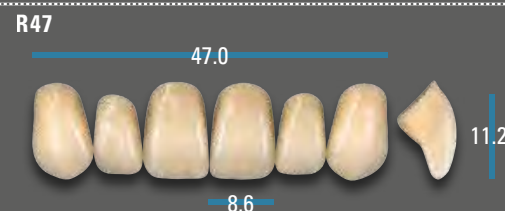



square

S46

46.3



8.5



10.3

Lower Anteriors

L33

33.2



5.0



8.6

L35

35.3



5.2



9.0

L37

37.3



5.2



9.4

L37L* (L37 with longer tooth neck)

37.3



5.2



9.4

L39

39.0



5.9



9.8

L41

41.2



6.2



10.2

VITAPAN PLUS Anterior	UJ / No.  mm	041 41.3	043 43.1	045 45.1	047 47.3	047L* 47.3	049 49.1	051 51.4	T42 42.0	T44 44.1	T46 46.2	T47 46.9	T48 48.1	T50 50.0	T54 54.4	R45 45.4	R47 47.0	R49 49.1	S46 46.3
	LJ / No.  mm	L33 33.2	L33 33.2	L35 35.3	L37 37.3	L37L* 37.3	L39 39.0	L41 41.2	L33 33.2	L33 33.2	L35 35.3	L37 37.3	L37 37.3	L39 39.0	L 41 41.2	L35 35.3	L37 37.3	L39 39.0	L35 35.3
VITAPAN LINGOFORM Posterior	UJ / No.  mm	21L 30.1	21L 30.1	22L 31.6	22L 31.6	22L 31.6	23L 34.2	24L 35.8	21L 30.1	21L 30.1	22L 31.6	22L 31.6	22L 31.6	24L 35.8	24L 35.8	22L 31.6	22L 31.6	23L 34.2	22L 31.6
	LJ / No.  mm	21L 30.7	21L 30.7	22L 32.1	22L 32.1	22L 32.1	23L 34.8	24L 36.4	21L 30.7	21L 30.7	22L 32.1	22L 32.1	22L 32.1	24L 36.4	24L 36.4	22L 32.1	22L 31.6	23L 34.8	22L 32.1

The information contained in the combination table is only intended as a guide, and can vary depending on the patient's individual situation. *long (047 or L37 with longer tooth neck)

VITAPAN
EXCELL

VITAPAN
PLUS

VITAPAN
LINGOFORM

VITA
PHYSIODENS

VITAPAN

VITAPAN
CUSPIFORM

VITAPAN
SYNOFORM

VITA
MFT

VITA VIONIC
VIGO

VITA VIONIC
FRAME

VITAPAN® LINGOFORM

Available in 27 x VITA SYSTEM 3D-MASTER® shades (0M3, 1M1 – 5M3) and 15 x VITA classical A1 – D4® shades (without B1)

Posterior

21L* 30.1 9.6 30.7 10.0	22L* 31.6 10.0 32.1 10.2	23L* 34.2 10.9 34.8 10.8
24L* 35.8 11.0 36.4 11.2		

*) Bleached shade 0M3 available exclusively for selected mould(s).

VITAPAN
EXCELL

VITAPAN
PLUS

VITAPAN
LINGOFORM

VITA
PHYSIODENS

VITAPAN

VITAPAN
CUSPIFORM

VITAPAN
SYNOFORM

VITA
MFT

VITA VIONIC
VIGO

VITA VIONIC
FRAME

VITA PHYSIODENS®

Available in 28 x VITA SYSTEM 3D-MASTER® shades (0M1, 0M3, 1M1 – 5M3) and 15 x VITA classical A1–D4® shades (without B1)

Upper Anteriors



01S* 	02L 	03M 	04L
05L 	06L 		
T6S** 	T2S* 	T1S 	T3M
T4M* 	T5M 	T6L 	T7M
T8L* 	T9L 		





rectangular

X1M*

46.3



8.8



11.0

X2L

46.2



8.7



11.6

X3L

47.8



9.0



11.8



square

Z1S

44.1



8.1



9.8

Z2S

47.5



8.7



10.1

Lower Anteriors

L1S*

33.0



4.9



8.8

L2M*

34.5




5.2



10.4

L3M*

36.3

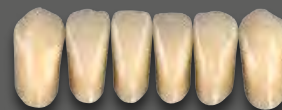

5.2



10.5

L4M*

35.5



5.4



9.4

L5M*

37.0



5.5



10.4

L6L

38.2




5.8



11.4

L7L

38.2

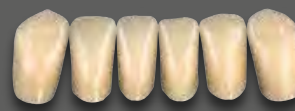

5.5



L8L

43.5



6.3



13.0

*) Bleached shade(s) 0M1 and 0M3 available exclusively for selected mould(s).

**) Mould(s) only available in VITA SYSTEM 3D-MASTER shade(s).

VITAPAN
EXCELL

VITAPAN
PLUS

VITAPAN
LINGOFORM

VITA
PHYSIODENS

VITAPAN

VITAPAN
CUSPIFORM

VITAPAN
SYNOFORM

VITA
MFT

VITA VIONIC
VIGO

VITA VIONIC
FRAME

VITA PHYSIODENS Anterior	UJ / No.  mm	01S 42.3	02L 45.7	03M 47.2	04L 49.2	05L 50.1	06L 51.3	T6S 41.0	T2S 44.9	T1S 45.3	T3M 45.6	T4M 45.6	T5M 46.2	T6L 47.3	T7M 47.5	T8L 49.0	T9L 49.6
	LJ / No.  mm	L1S 33.0	L2M 34.5	L3M 36.3	L6L 38.2	L6L 38.2	L8L 43.5	L1S 33.0	L2M 34.5	L1S 33.0	L3M 36.3	L4M 35.5	L3M 36.3	L5M 37.0	L7L 38.2	L5M 37.0	L6L 38.2
VITA PHYSIODENS Posterior	UJ / No.  mm	21E 30.5	22E 32.9	23E 33.5	24E 34.8	24E 34.8	25E 36.7	21E 30.5	21E 30.5	20E 29.4	22E 32.9	22E 32.9	23E 33.5	23E 33.5	23E 33.5	24E 34.8	25E 36.7
	LJ / No.  mm	21E 32.7	22E 33.7	23E 34.6	24E 36.1	24E 36.1	25E 37.3	21E 32.7	21E 32.7	20E 31.0	22E 33.7	22E 33.7	23E 34.6	23E 34.6	23E 34.6	24E 36.1	25E 37.3

VITA PHYSIODENS Anterior	UJ / No.  mm	X1M 46.3	X2L 46.2	X3L 47.8	Z1S 44.1	Z2S 47.5
	LJ / No.  mm	L3M 36.3	L4M 35.5	L5M 37.0	L1S 33.0	L5M 37.0
VITA PHYSIODENS Posterior	UJ / No.  mm	23E 33.5	23E 33.5	24E 34.8	21E 30.5	22E 32.9
	LJ / No.  mm	23E 34.6	23E 34.6	24E 36.1	21E 32.7	22E 33.7

The information contained in the combination table is only intended as a guide, and can vary depending on the patient's individual situation.

VITA PHYSIODENS®

Available in 27 x VITA SYSTEM 3D-MASTER® shades (0M3, 1M1 – 5M3) and 15 x VITA classical A1 – D4® shades (without B1)

Posterior

20E 29.4 8.9 31.0 8.3	21E* 30.5 9.5 32.7 8.7	22E* 32.9 9.5 33.7 9.0
23E* 33.5 11.1 34.6 9.8	24E* 34.8 11.8 36.1 9.8	25E 36.7 11.2 37.3 9.5

*) Bleached shade 0M3 available exclusively for selected mould(s).


013 41.7 7.9 9.5	025 42.8 8.1 9.9	034 43.3 8.0 9.7	035 43.2 8.0 9.8
043 43.6 8.2 9.4	086 47.1 8.9 9.9	097 49.2 9.0 11.5	098* 48.1 9.0 11.8
099 52.5 9.8 12.5			
T36* 43.7 8.1 10.1	T53 44.6 8.1 9.2	T56 44.4 8.3 10.0	T66 44.6 8.3 10.2
T67 45.1 8.4 10.0	T76* 45.9 8.2 10.4	T77* 46.2 8.5 10.6	T88* 47.1 8.7 10.6





rectangular


square


T98

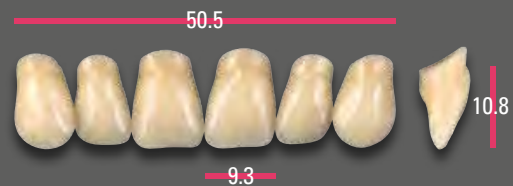


T99

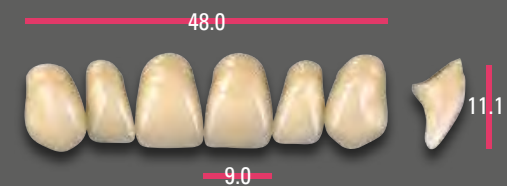


X13

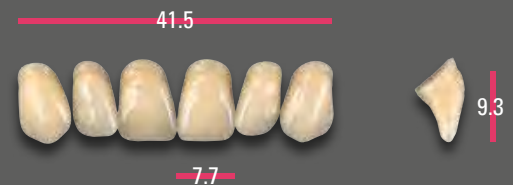


X66

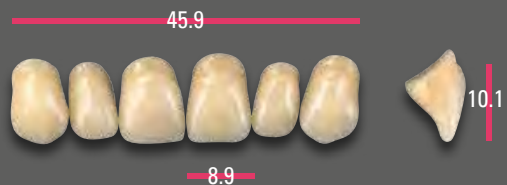


X77

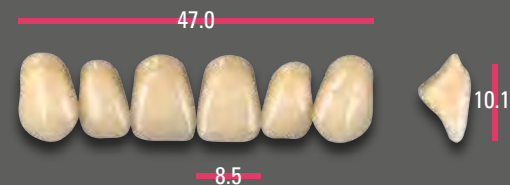


X87

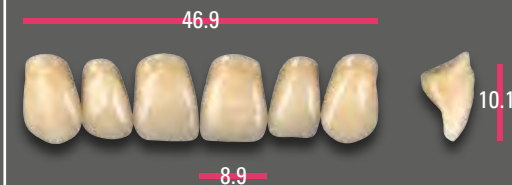


X96

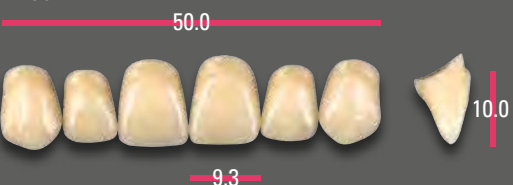


X99

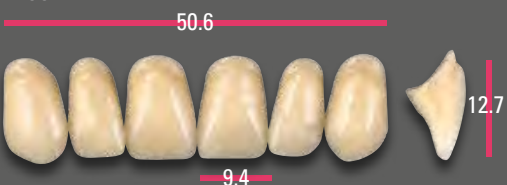


Z51

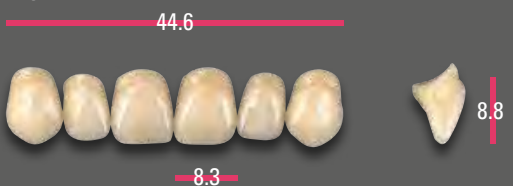


Z61

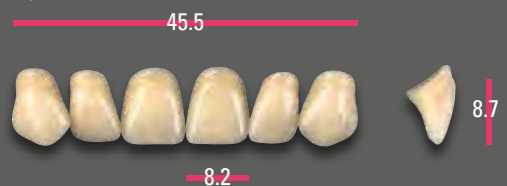


Z74

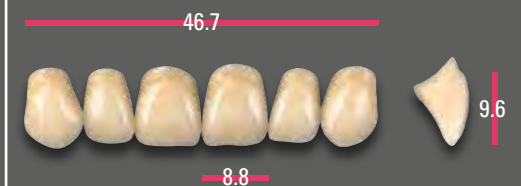


Z84

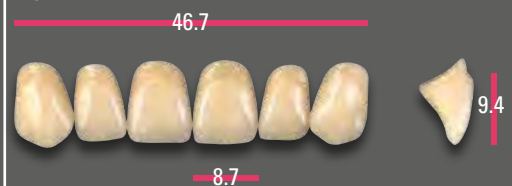

Z85

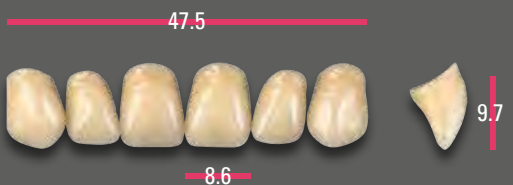

Z97

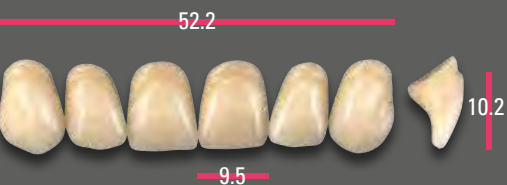

VITAPAN
EXCELL

VITAPAN
PLUS

VITAPAN
LINGOFORM

VITA
PHYSIODENS

VITAPAN

VITAPAN
CUSPIFORM





VITAPAN
SYNOFORM

VITA
MFT

VITA VIONIC
VIGO

VITA VIONIC
FRAME

Lower Anteriors

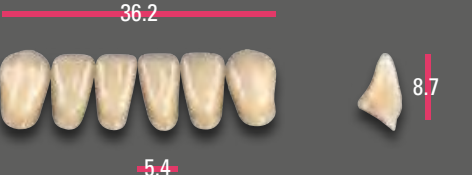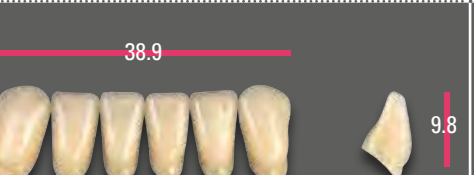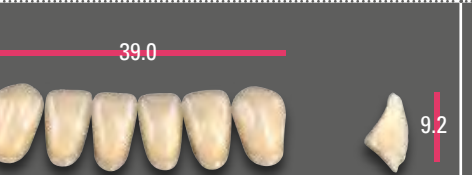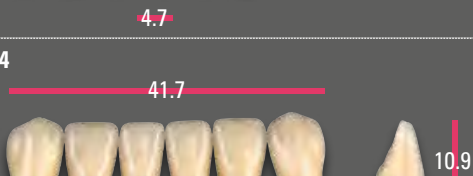Fully anatomical incisal edges	L3 	L5* 	L5L 	L7* 
	L9 	L11* 	L13 	L15 
Abraded incisal edges	L4* 	L8 	L10 	L12 
	L14 			

*) Bleached shade(s) 0M1 and 0M3 available exclusively for selected mould(s).

VITAPAN Anterior	UJ / No.  mm	013 41.7	025 42.8	034 43.3	035 43.2	043 43.6	086 47.1	097 49.2	098 48.1	099 52.5	T36 43.7	T53 44.6	T56 44.4	T66 44.6	T67 45.1	T76 45.9	T77 46.2	T88 47.1	T98 50.5	T99 48.0
	LJ / No.  mm	L4 32.7	L3 33.4	L3 33.4	L3 33.4	L5 35.3	L11 36.9	L8 38.9	L13 37.0	L14 41.7	L4 32.7	L5 35.3	L5 35.3	L5 35.3	L4 32.7	L7 36.2	L11 36.9	L11 36.9	L15 38.0	L11 36.9
VITAPAN CUSPIFORM Posterior	UJ / No.  mm	40C 28.1	40C 28.1	41C 29.9	41C 29.9	41C 29.9	42C 31.7	44C 32.8	42C 31.7	45C 34.8	40C 28.1	41C 29.9	41C 29.9	42C 31.7	42C 31.7	42C 31.7	42C 31.7	44C 32.8	45C 34.8	44C 32.8
	LJ / No.  mm	40C 29.5	40C 29.5	41C 31.3	41C 31.3	41C 31.3	42C 33.4	44C 34.6	42C 33.4	45C 36.3	40C 29.5	41C 31.3	41C 31.3	42C 33.4	42C 33.4	42C 33.4	42C 33.4	44C 34.6	45C 36.3	44C 34.6
VITAPAN SYNOFORM Posterior	UJ / No.  mm	10S 29.0	10S 29.0	11S 30.0	11S 30.0	11S 30.0	12S 31.3	14S 33.1	12S 31.3	14S 33.1	10S 29.0	11S 30.0	11S 30.0	12S 31.3	12S 31.3	12S 31.3	12S 31.3	14S 33.1	14S 33.1	14S 33.1
	LJ / No.  mm	10S 30.5	10S 30.5	11S 31.9	11S 31.9	11S 31.9	12S 32.7	14S 35.4	12S 32.7	14S 35.4	10S 30.5	11S 31.9	11S 31.9	12S 32.7	12S 32.7	12S 32.7	12S 32.7	14S 35.4	14S 35.4	14S 35.4

VITAPAN Anterior	UJ / No.  mm	X13 41.5	X66 45.9	X77 47.0	X87 46.9	X96 50.0	X99 50.6	Z51 44.6	Z61 45.5	Z74 46.7	Z84 46.7	Z85 47.5	Z97 52.2
	LJ / No.  mm	L4 32.7	L9 36.1	L11 36.9	L11 36.9	L10 39.0	L15 38.0	L3 33.4	L3 33.4	L9 36.1	L9 36.1	L13 37.0	L12 41.0
VITAPAN CUSPIFORM Posterior	UJ / No.  mm	41C 29.9	42C 31.7	42C 31.7	44C 32.8	44C 32.8	45C 34.8	43C 31.0	43C 31.0	42C 31.7	42C 31.7	44C 32.8	45C 34.8
	LJ / No.  mm	41C 31.3	42C 33.4	42C 33.4	44C 34.6	44C 34.6	45C 36.3	43C 32.3	43C 32.3	42C 33.4	42C 33.4	44C 34.6	45C 36.3
VITAPAN SYNOFORM Posterior	UJ / No.  mm	11S 30.0	12S 31.3	12S 31.3	14S 33.1	14S 33.1	14S 33.1	13S 31.3	13S 31.3	12S 31.3	12S 31.3	14S 33.1	14S 33.1
	LJ / No.  mm	11S 31.9	12S 32.7	12S 32.7	14S 35.4	14S 35.4	14S 35.4	13S 33.9	13S 33.9	12S 32.7	12S 32.7	14S 35.4	14S 35.4


The information contained in the combination table is only intended as a guide, and can vary depending on the patient's individual situation.

VITAPAN® CUSPIFORM

Available in 27 x VITA SYSTEM 3D-MASTER® shades (0M3, 1M1 – 5M3) and 15 x VITA classical A1 – D4® shades (without B1)

Posterior


40C

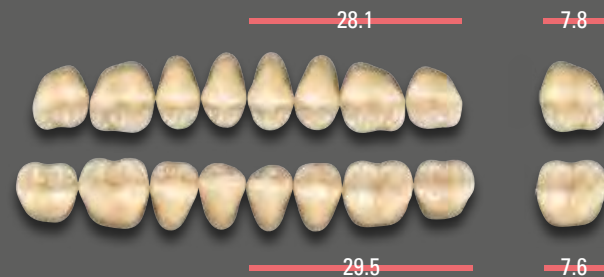


41C*




42C*

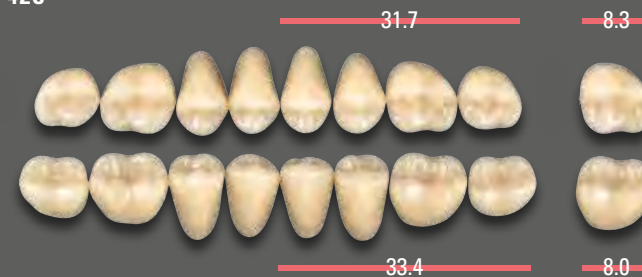


43C

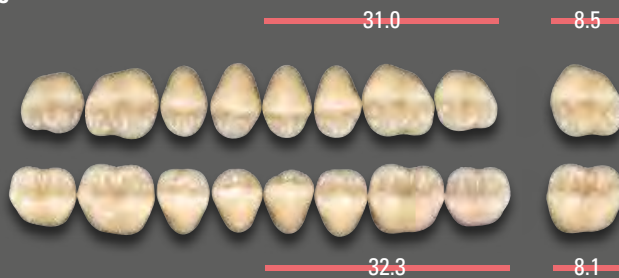

44C*





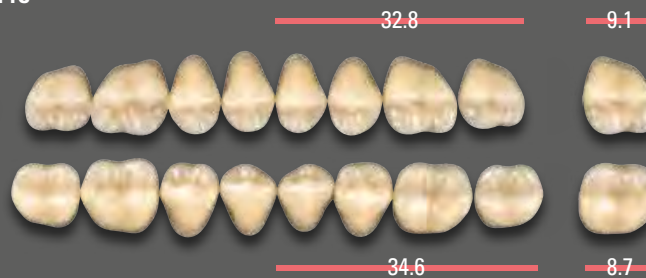


45C

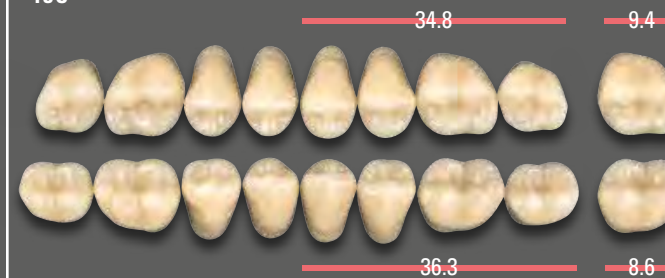

*) Bleached shade 0M3 available exclusively for selected mould(s).

Posterior

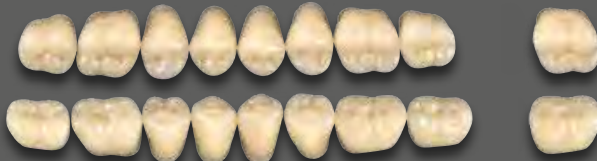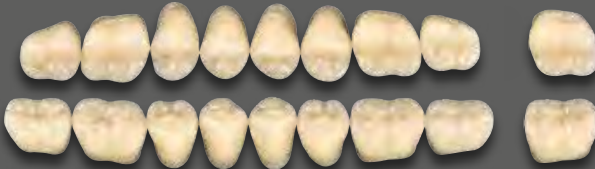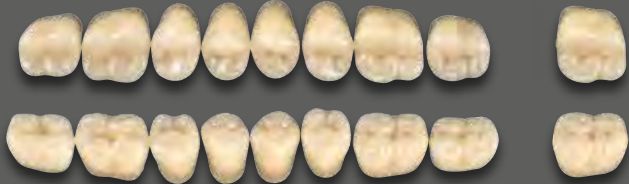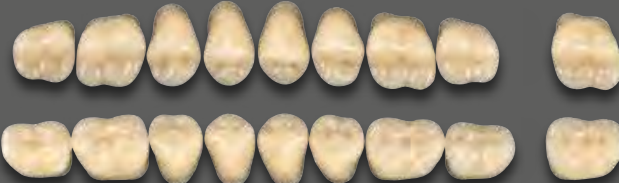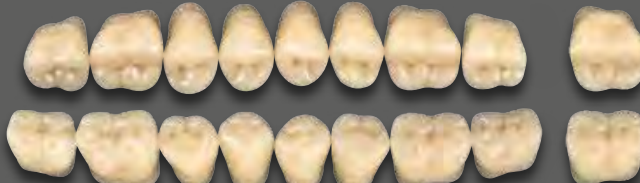10S 29.0 7.5  30.5 6.8	11S 30.0 7.8  31.9 7.3	12S 31.1 8.7  32.7 7.8
13S 31.3 8.3  33.9 7.4	14S 33.1 9.3  35.4 8.3	


Upper Anteriors



ovoid

040

40.0

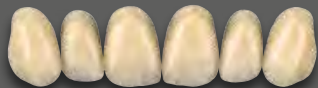

7.5

10.0

044

43.9



8.1

10.2

049

49.2



9.0


11.8



triangular

T41

40.5

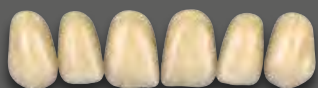

7.5

9.4

T43

43.3



8.2

10.3

T46

46.1



8.5

10.2

T49

48.8



9.0


11.8



rectangular

R42

42.1

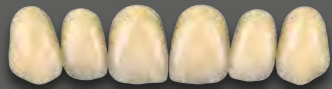

7.9

9.3

R45

44.5



8.2

10.3

R47

46.6



8.5

11.0



square

S47

46.8



8.9

10.8

S50


49.7



9.2

12.0

Lower Anteriors

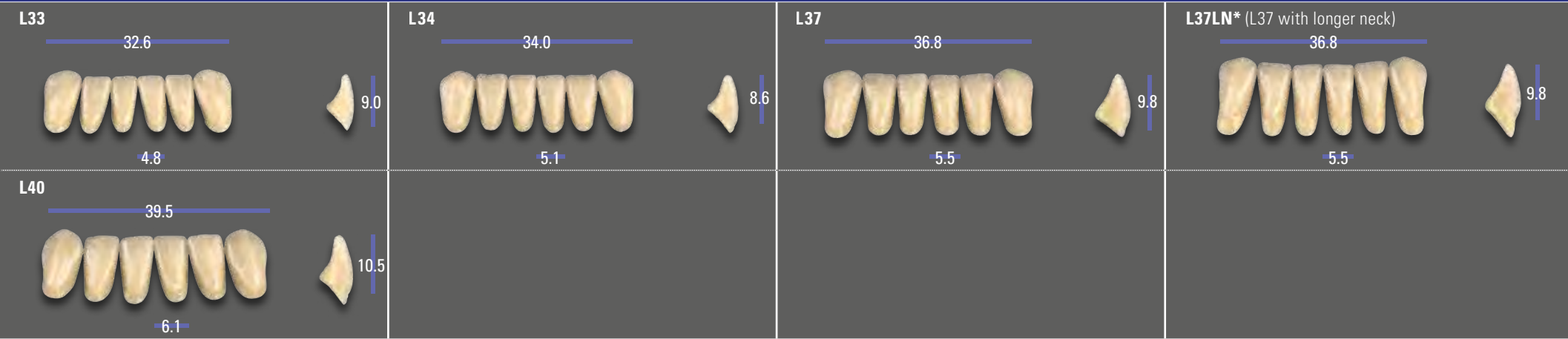


VITA MFT Anterior	UJ / No.  mm	040 40.0	044 43.9	049 49.2					R42 42.1	R45 44.5	R47 46.6	S47 46.8	S50 49.7
	LJ / No.  mm	L33 32.6	L34 34.0	L40 39.5	L33 32.6	L34 34.0	L37 36.8	L40 39.5	L34 34.0	L37 36.8	L37 36.8	L40 39.5	L40 39.5
VITA MFT Posterior	UJ / No.  mm	PU29 28.8	PU29 28.8	PU33 33.0	PU29 28.8	PU31 30.6	PU31 30.6	PU33 33.0	PU29 28.8	PU31 30.6	PU33 33.0	PU33 33.0	PU33 33.0
	LJ / No.  mm	PL29 29.6	PL29 29.6	PL33 33.6	PL29 29.6	PL31 31.1	PL31 31.1	PL33 33.6	PL29 29.6	PL31 31.1	PL33 33.6	PL33 33.6	PL33 33.6


The information contained in the combination table is only intended as a guide, and can vary depending on the patient's individual situation. *long (O47 or L37 with longer tooth neck)

Posterior


PU29 – PL29

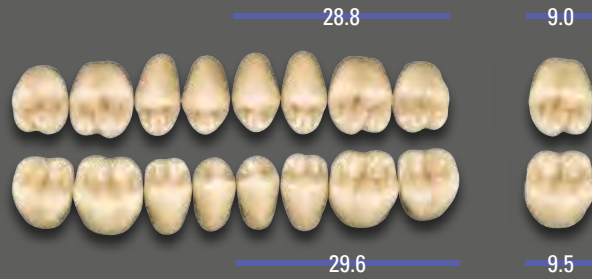

PU31 – PL31

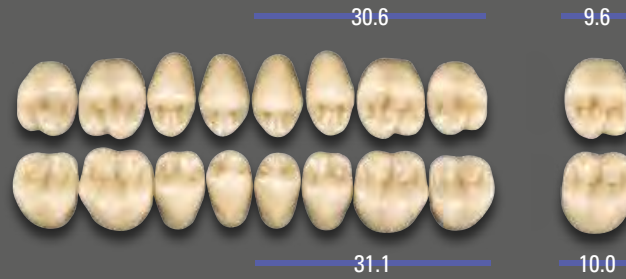

PU33 – PL33

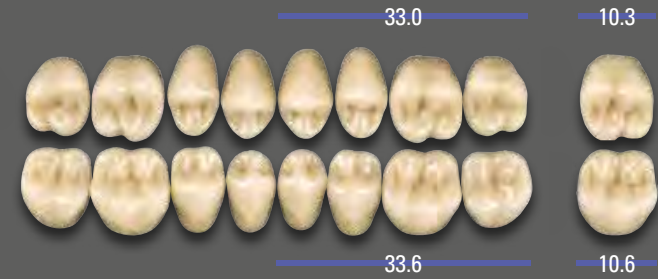

VITAPAN
EXCELL

VITAPAN
PLUS

VITAPAN
LINGOFORM

VITA
PHYSIODENS

VITAPAN

VITAPAN
CUSPIFORM

VITAPAN
SYNOFORM

VITA
MFT

VITA VIONIC
VIGO

VITA VIONIC
FRAME

Upper Anteriors

 ovoid	043  43.3 7.9 9.3	045  44.5 8.3 9.7		
	T44  44.0 8.1 9.9	T46  46.2 8.2 10.2	T48  48.3 8.7 10.8	T50  49.9 9.2 10.8
	R47  47.0 8.6 9.8	R49  49.0 8.9 10.7		
 triangular				
 rectangular				

Lower Anteriors

L33  33.2 4.9 7.2	L35  35.2 5.2 7.5	L37  37.2 5.5 7.8	L39  39.2 5.8 8.1
---	--	---	---

VITA VIONIC VIGO®

Available in 6 x VITA classical A1–D4®-/Bleach-shades: 0M1, A1, A2, A3, A3.5, B3, D3

Posterior











VITA VIONIC VIGO Anterior	UJ / No.  mm	R47 47.0	R49 49.0	O45 44.5	T50 49.9	T46 46.2	O43 43.3	T44 44.0	T48 48.3
	LJ / No.  mm	L37 37.2	L39 39.2	L35 35.2	L39 39.2	L35 35.2	L33 33.2	L33 33.2	L37 37.2
VITA VIONIC VIGO Posterior	UJ / No.  mm	22L 31.8	23L 34.3	22L 31.8	24L 35.9	22L 31.8	21L 30.3	21L 30.3	22L 31.8
	LJ / No.  mm	22L 32.6	23L 35	22L 32.6	24L 36.9	22L 32.6	21L 31	21L 31	22L 32.6

VITAPAN EXCELL® DD FRAME

Available in 6 x VITA classical A1–D4® shades (A1, A2, A3, A3.5, B3, D3)

Upper Anteriors

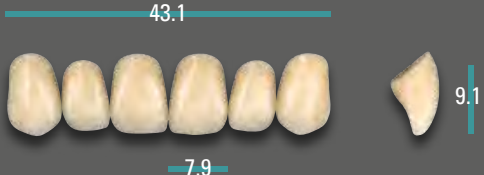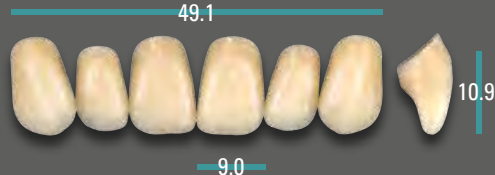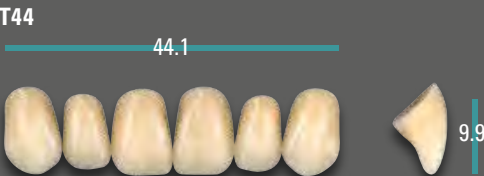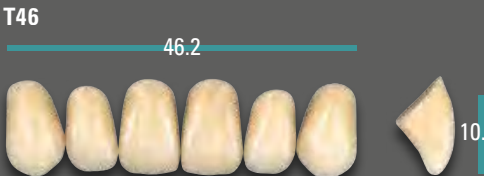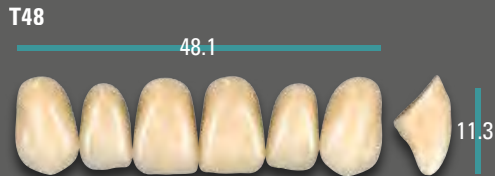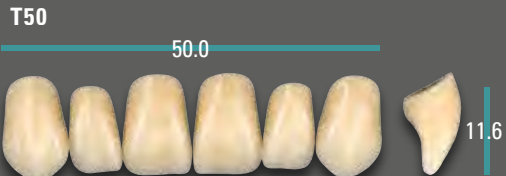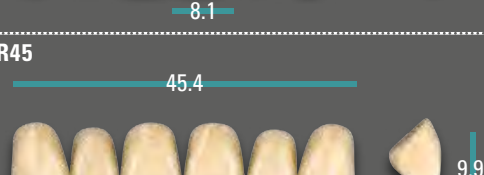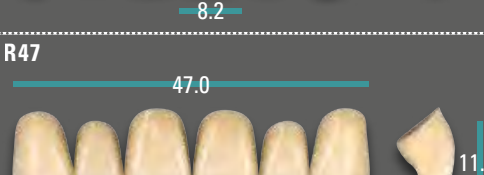 ovoid	043 	045 	049 	
	T44 	T46 	T48 	T50 
	R45 	R47 		

Lower Anteriors

L33 	L35 	L37 	L39 
---	--	---	---

VITAPAN® LINGOFORM DD FRAME


Available in 6 x VITA classical A1–D4® shades (A1, A2, A3, A3.5, B3, D3)

Posterior



VITAPAN EXCELL Anterior	UJ / No.  mm	043 43.1	045 45.1	049 49.1	T44 44.1	T46 46.2	T48 48.1	T50 50.0	R45 45.4	R47 47.0
	LJ / No.  mm	L33 33.2	L35 35.3	L39 39.0	L33 33.2	L35 35.3	L37 37.3	L39 39.0	L35 35.3	L37 37.3
VITAPAN LINGOFORM Posterior	UJ / No.  mm	21L 30.1	22L 31.6	23L 34.2	21L 30.1	22L 31.6	22L 31.6	24L 35.8	22L 31.6	22L 31.6
	LJ / No.  mm	21L 30.7	22L 32.1	23L 34.8	21L 30.7	22L 32.1	22L 32.1	24L 36.4	22L 32.1	22L 32.1

For more information on **VITA PROSTHETIC SOLUTIONS**, see:
www.vita-zahnfabrik.com/prosthetics

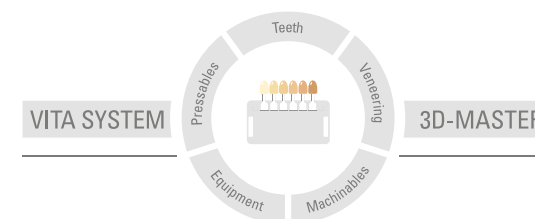

Please note: Our products must be used in accordance with the instructions for use. We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of this information: 2021-06

After the publication of this information for use any previous versions become obsolete. The current version can be found at www.vita-zahnfabrik.com

VITA Zahnfabrik has been certified and the following products bear the mark:

CE 0124:

VITAPAN® product range, **VITA PHYSIODENS**®, **VITA MFT**®, **VITA VIONIC**® product range



VITA

 VITA Zahnfabrik H. Rauter GmbH & Co.KG
Spitalgasse 3 · D-79713 Bad Säckingen · Germany
Tel. +49(0)7761/562-0 · Fax +49(0)7761/562-299
Hotline: Tel. +49(0)7761/562-222 · Fax +49(0)7761/562-446
www.vita-zahnfabrik.com · info@vita-zahnfabrik.com
 facebook.com/vita.zahnfabrik